Apteq Korvakuumemittari

Käyttöohje

RA600

APTEQ

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EMC guidance and manufacturer’s declaration

Electromagnetic Compatibility Information

- This device needs to be installed and put into service in accordance with the information provided in the user manual.
- WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the RA600, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result.

If higher IMMUNITY TEST LEVELS than those specified in Table 9 are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in 8.10.

Manufacturer’s declaration–electromagnetic immunity			
The RA600 is intended for use in the electromagnetic environment specified below. The customer or the user of the RA600 should assure that is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment–guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0,15 MHz – 80 MHz <p>6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz</p> <p>80 % AM at 1 kHz</p>	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the RA600 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: <p>d = 1,2 √P, d = 1,2 √P 80MHz to 800 MHz, d = 2,3 √P 800MHz to 2,7 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
		Not applicable	
Radiated RF IEC 61000-4-3	10 V/m <p>80 MHz – 2,7 GHz</p> <p>80 % AM at 1 kHz</p>	10 V/m <p>80 MHz – 2,7 GHz</p> <p>80 % AM at 1 kHz</p>	


NOTE1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people


More information on EMC compliance of the device can be obtained from APTEQ website: www.aptEq.fi

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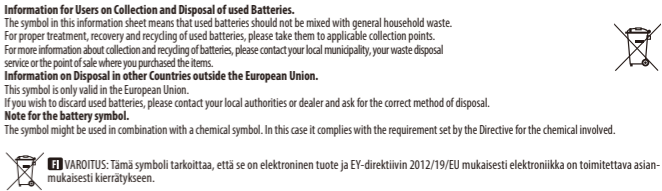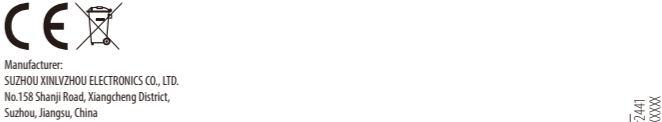WARRIORS
Manganese dioxide Lithium Battery
Portable cell (non-zinc-air button cell)
MODEL NO.: CR2032
CADMIUM, MERCURY AND LEAD FREE
Nominal Capacity: 210mAh
Nominal Voltage: 3.0V

Manufacturer:
SUZHOU XINLVZHOU ELECTRONICS CO., LTD.
No.158 Shanji Road, Xiangcheng District,
Suzhou, Jiangsu, China

RA600

aptEq.fi

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Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

Information on Disposal in other Countries outside the European Union.

If you wish to discard used batteries, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol.

The symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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EMC guidance and manufacturer's declaration

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Manufacturer's declaration-electromagnetic immunity			
The RA600 is intended for use in the electromagnetic environment specified below. The customer or the user of the RA600 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0,15 MHz – 80 MHz 6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the RA600 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: d = 1,2 √P, d = 1,2 √P 80MHz to 800 MHz, d = 2,3 √P 800MHz to 2,7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol:
	80 % AM at 1 kHz	Not applicable	
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	

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More information on EMC compliance of the device can be obtained from APTEQ website: www.apteq.fi

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Inköpsdatum:

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Köparen:

Information for Users on Collection and Disposal of used Batteries.

The symbol in this information sheet means that used batteries should not be mixed with general household waste.

For proper treatment, recovery and recycling of used batteries, please take them to applicable collection points.

For more information about collection and recycling of batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

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This symbol is only valid in the European Union.

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Fennogate Finland Oy
Rossmax Swiss GmbH